

**NAUGRA**<sup>®</sup>

Email : sales@naugramedical.com  
Phone: 9896600003

**Product Name :**  
Fluoride Oxalate Tubes (MOQ: 100,000 Pcs Assorted Size)

**Product Code :**  
Blood0011



**Description :**

Fluoride Oxalate Tubes (MOQ: 100,000 Pcs Assorted Size) This PET Fluoride/Oxalate Glucose Blood Collection tube is utilized for glucose determinations. Glucose values in unpreserved blood samples deteriorate swiftly after collection as glucose is used by the blood cells. The Fluoride will stop enzymatic activity while oxalate acts as an anti-coagulant. Mixing recommendation: Fluoride/Oxalate tubes should be gently inverted 180° and back 8-10 times. Centrifugation instructions: 1300g for 10 minutes at 25° C room temperature

**NAUGRA**<sup>®</sup>

**Medical Hospital Equipments Manufacturers**

Website: Medical Hospital Equipments Manufacturers,